**Bedienungsanleitung**  
**User Manual**  
**Instructions d'utilisation**  
**Manuale d'uso**  
**Manual de instrucciones**  
**Gebruiksaanwijzing**  
**Betjeningsvejledning**  
**Bruksanvisning**  
**Bruksanvisning**  
**Käyttöohje**  
**Instrukcja obsługi**  
**Návod k obsluze**  
**取扱説明書**

DE

EN

FR

IT

ES

NL

DA

SV

NO

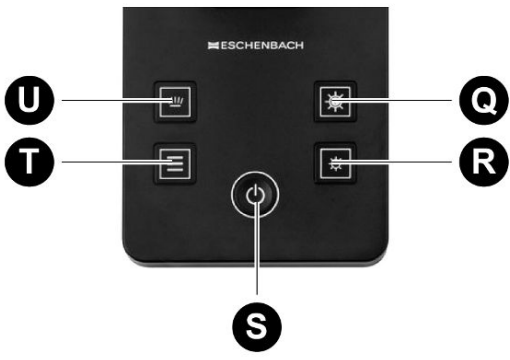
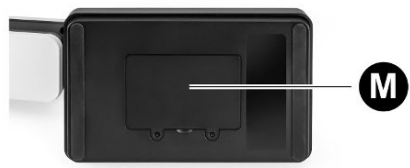
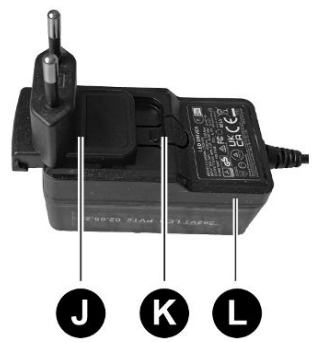
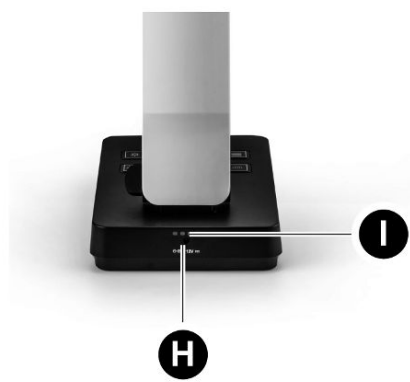
FI

PL

CS

JA

**elumentis LED**

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<b>Deutsch .....</b>	<b>4</b>
<b>English .....</b>	<b>16</b>
<b>Français.....</b>	<b>27</b>
<b>Italiano .....</b>	<b>40</b>
<b>Español .....</b>	<b>51</b>
<b>Nederlands .....</b>	<b>63</b>
<b>Dansk.....</b>	<b>74</b>
<b>Svenska.....</b>	<b>85</b>
<b>Norsk .....</b>	<b>96</b>
<b>Suomalainen .....</b>	<b>107</b>
<b>Polski .....</b>	<b>118</b>
<b>Česky .....</b>	<b>131</b>
<b>日本語.....</b>	<b>142</b>

# Deutsch

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# English

## About this manual

This manual applies to the following products:

	Nr. No.
Desk lamp elumentis LED	16040
Desk lamp elumentis LED with enclosed rechargeable battery	16040 160404

Read through this manual carefully before using the product described in it. Keep the manual in a safe place. If you pass on the product, give the recipient this manual too.

The following signs and symbols are used in this manual and/or on the product:

Symbol	Meaning
	General warning of a hazard
	Refer to instruction manual
	Dispose of the product according to the local guidelines
	The product is recyclable

## Safety instructions

The product is designed and built to state-of-the-art standards and complies with the relevant European directives. Nonetheless, residual hazards exist. To avoid hazards, you must follow the safety instructions. Make sure that other people are also aware of these hazards.

Children can injure themselves when handling the product. Never allow unsupervised children to play with packaging material and the product.

**Risk of electric shock**

- Use the product only with the power adaptor supplied with it or with the (optionally available) rechargeable battery. If the housing, power adaptor or cable are damaged, stop using the product.
- Do not wind the power cord around the product.
- Do not bend the power cord.
- Do not use the product near containers filled with water and do not allow the product to come into contact with water.

**Risk when handling the device**

- Do not look directly at or into the light source for long periods.
- Lay the power cord so that it does not pose a tripping hazard.
- Make sure you do not trap your fingers when folding and unfolding moving parts.

**Risk of property damage**

- To avoid overheating, do not cover the lamp head.
- Never place the product near heat sources or in direct sunlight.
- Protect your desk lamp from impact or shock, moisture and excessive heat.

**Risk when handling the rechargeable battery**

- Do not open or dismantle the rechargeable battery.
- Do not expose the rechargeable battery to heat or fire. Avoid storage in direct sunlight.
- Keep the rechargeable battery clean and dry.
- Do not store the rechargeable battery without packaging in containers in which short-circuits can be caused with loose metallic objects.
- Keep the rechargeable battery out of reach of children.
- Do not remove the rechargeable battery from the original packaging until you want to use it.
- Do not store the rechargeable battery in the discharged state for longer than 1 month.

## Intended use

The product is intended for illuminating the reading and working area. It is only suitable for indoor use in dry rooms.

Only use the product as described in this manual. Any other use is considered improper and can lead to property damage or personal injury. Do not make any changes to the construction of the product. No liability is assumed for damage caused by improper or incorrect use.

## Product description

### Light

A Lamp head	B Joint for swivelling the lamp head
C Stand	D Base joint of the lamp arm
E Lamp arm	F Head joint of the lamp arm
G Set screw for readjusting the middle joint on the lamp head	H Socket for plugging in the power cord
I Charge level indicator (4 red LEDs) <ul style="list-style-type: none"><li>– Battery fully charged: all 4 LEDs are lit</li><li>– Low battery charge: 1 LED is lit</li><li>– Critical battery charge: 1 LED flashes</li></ul>	
The charge level indicator does not function if the lamp is used without the battery.	
M Battery compartment cover with Phillips screws	N Battery compartment

## Controls

Q	 [Increase brightness] button	R	 [Decrease brightness] button
S	 [On/Off] button	T	 [Colour temperature] button
U	 [Ambient light] button		

## Power adaptor

J	Mains plug adaptor	K	[PUSH] button
L	Power adaptor		

## Rechargeable battery

P	Grip tab	O	Notch
---	----------	---	-------

## Operation

### Unpacking the product

1. Unpack the product and check it carefully for damage.
2. Keep the packaging material for transport and possible returns.

### Positioning the product

- Place the product on a level and stable surface.

### Adjusting the lamp head and lamp arm

1. You can lift or lower the lamp head [A] and the lamp arm [E] by 90°, swivel the lamp head by 90° to the right or left and swivel the lamp head by 45° axially. Adjust the lamp head and the lamp arm until the position of the lamp meets your needs.
2. If the set position of the lamp head is not maintained or is too stiff, readjust the head joint [F] or the base joint [D] of the lamp arm at the toggle screw [B].
3. If the holding force of the middle joint of the lamp head weakens: use an Allen key to retighten the set screw [G].

## Inserting the rechargeable battery

Only insert the manufacturer-supplied rechargeable battery with model number RRC2040 in the lamp!

1. Make sure that the lamp is switched off and disconnected from the power supply.
2. Use a Phillips screwdriver to undo the two screws in the battery compartment cover *[M]* on the underside of the lamp base.
3. Remove the battery compartment cover.
4. Insert the rechargeable battery supplied with the lamp or purchased as an accessory in the battery compartment *[N]*.
  - ⇒ The notch *[O]* on the rechargeable battery is positioned exactly on the web of the battery compartment. The rechargeable battery latches into the battery compartment.
5. Place the battery compartment cover on the battery compartment.
6. Use the two screws to fasten the battery cover.
  - ⇒ The lamp can be used in battery mode.
  - ⇒ In battery mode, after switching on the lamp and if the charge level changes, the charge level of the battery is indicated for 30 seconds at the lamp's charge level indicator. If the battery level is critical, the last LED starts to flash.

## Preparing the power adaptor

1. Select the mains plug adaptor *[J]* that matches your mains socket (EU, US, UK, AUS).
2. Fit the mains plug adaptor on the power adaptor *[L]*. To do so, place the side detents of the mains plug adapter into the notches of the power adaptor and push the adapter towards the front until it latches into position with a click.
  - ⇒ The power adaptor is ready for use.
3. To detach the mains plug adaptor, press the *[PUSH]* button *[K]* on the power adaptor.

## Connecting the lamp to the power supply

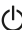
✓ The mains plug adaptor is connected to the power adaptor.

1. Insert the angle plug of the supplied power adaptor [L] into the socket at the rear of the lamp's stand.
2. Insert the mains plug of the power adaptor into a properly connected mains socket.

⇒ The lamp can be used in mains mode.

⇒ If a rechargeable battery is inserted: the rechargeable battery is charged. During charging, the charge level of the battery is indicated at the lamp's charge level indicator. When the rechargeable battery is fully charged, the charge level indicator light goes out.

## Switching the lamp on and off

1. Press the [On/Off] button  [S] to switch on the light. Slight heating in the area of the lit LEDs is normal.
2. Press the [On/Off] button  [S] again to switch off the light.

## Adjusting the light

### Adjusting the brightness

1. Short press the [Increase brightness] button  [Q] repeatedly to increase the brightness in 5 steps. Keep the button pressed to increase the brightness continuously.
2. Briefly press the [Decrease brightness] button  [R] repeatedly to decrease the brightness in 5 steps. Keep the button pressed to reduce the brightness continuously.

The brightness is set separately for each colour temperature. The respective brightness is retained when the desk lamp is switched off and when the colour temperatures are switched.

## Setting the colour temperature

You can set the following colour temperatures:

- 2700 K: warm white light
- 4800 K: neutral white light
- 6400 K: cold white light
- Press the *[Colour temperature]* button  *[T]* repeatedly until the required colour temperature is reached.

## Adjusting the ambient light

In addition to the main light, you can also switch on an ambient light. You can set the following combinations of main light and ambient light:

- Main light switched on / ambient light switched off
- Main light switched on / ambient light switched on
- Main light switched off / ambient light switched on
- Press the *[Ambient light]* button  *[U]* repeatedly until the required combination is reached.

## Disconnecting the lamp from the power supply

- Pull the mains plug out of the mains socket.
- ⇒ The lamp is disconnected from the power supply.
- ⇒ If a rechargeable battery is inserted: the lamp can continue to be used in battery mode.

## Removing the rechargeable battery

1. Make sure that the lamp is switched off and disconnected from the power supply.
2. Use a Phillips screwdriver to undo the two screws in the battery compartment cover *[M]* on the underside of the lamp base.
3. Remove the battery compartment cover.
4. Grip the grip tab *[P]* and pull the rechargeable battery out of the battery compartment *[N]*.
5. Place the battery compartment cover on the battery compartment.
6. Use the two screws to fasten the battery cover.

## Technical data

EN

### Light

Eschenbach product code	16040		
<b>Light source</b>	<b>Colour temperature</b>	<b>Max. usable luminous flux</b>	<b>Energy efficiency class</b>
Main light	2700 K	220 lm	D
	4800 K	1000 lm	E
	6400 K	640 lm	D
Ambient light	2700 K	170 lm	E
Dimmable	yes		
Built-in dimmer	yes		
Illuminant life	25000 h		
<b>Power supply</b>			
External AC adaptor	100 ... 240 V, 50/60 Hz		
Power consumption	Lamp operation:	10 W	
	Lamp operation + rechargeable battery charging:	24 W	
<b>Mechanical structure</b>			
Protection class	IP20		
Dimensions (L × W × H) (lamp arm vertical, lamp-shade horizontal)	271 × 130 × 540 mm		
Weight (without power cord, without rechargeable battery)	1940 g		
Cord length	2.5 m		
<b>Ambient conditions</b>			
Operating temperatures	0 °C ... 40 °C		

This lamp contains light sources in energy efficiency classes D/E (main light) and E (ambient light).

The lamp is preset as follows: main light 6400 K, 640 lm, ambient light switched off.

The light sources in this lamp may only be replaced by the manufacturer, their customer service or a similarly qualified person.

### Rechargeable battery

Eschenbach product code	160404
Type number	RRC2040
Type of rechargeable battery	Rechargeable lithium-ion battery
Rated voltage	10.80 V
Capacity	3.35 Ah
Max. charging current	2.35 A
Max. charging voltage	12.60 V
Max. discharge current	4.00 A
Charging time in the device	approx. 4.5 h
Operating time (min.)	4 h
<b>Mechanical structure</b>	
Protection class	IP40
Dimensions (L×W×H)	85 × 59 × 22 mm
Weight	170 g
<b>Ambient conditions</b>	
Operating temperatures	Discharging: -20 ... 60 °C
	Charging: 0 ... 45 °C
Storage temperatures	<1 year: -20 ... 20 °C
	<3 months: -20 ... 45 °C
	<1 month: -20 ... 50 °C

## Cleaning and care

1. Make sure that the lamp is switched off and disconnected from the power supply.
2. Wipe the surface of the lamp with a damp cloth. Do not use soap solutions containing plasticisers, alcohol-based solvents or abrasive cleaning products.

## Disposal

### Disposal of electrical equipment

This product is subject to the currently valid version of the European Directive on Waste Electrical and Electronic Equipment. Do not dispose of the product in the normal household waste bin.

Dispose of the product or parts of it through a certified disposal facility or your municipal disposal facility. Note and comply with the currently applicable regulations. If in doubt, contact your disposal management facility. Dispose of all packaging materials in an environmentally friendly manner.

### Disposal of batteries and rechargeable batteries

Do not dispose of batteries and rechargeable batteries in the household waste bin.

To dispose of the device, remove the batteries or rechargeable batteries from the device and hand them in to an official collection point.

## Declaration of Conformity

The product meets all requirements of the relevant EU directives and the national regulations. We confirm this with the CE marking. You can retrieve the declaration of conformity for this product on the following website: <http://www.eschenbach-vision.com/de-DE/konformitaetserklaerungen>.

## Guarantee

Within the parameters of the applicable statutory provisions, Eschenbach Optik GmbH provides a guarantee for the function of the product described in this manual with regard to any faults arising due to manufacturing defects or material defects. In the event of damage due to improper handling, including damage from a fall or impact, no guarantee claims can be made. The guarantee can only be claimed by presenting proof of purchase.

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