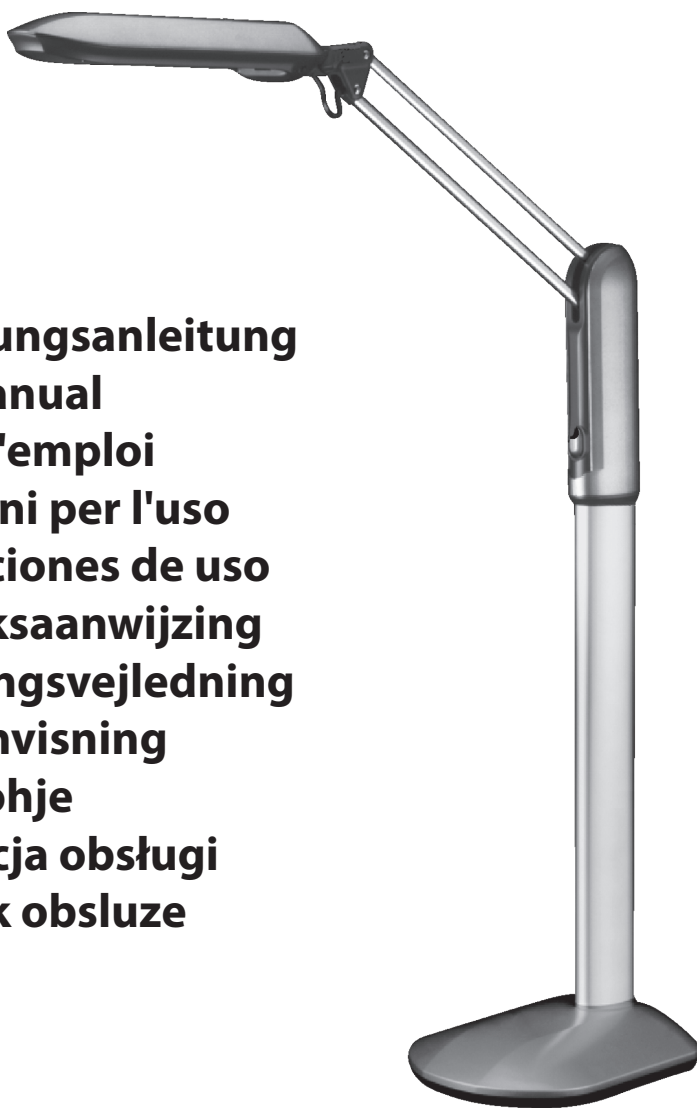
pure visionLED XL

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User Manual

Mode d'emploi

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
Bruksanvisning

Käyttöohje

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 **ESCHENBACH**

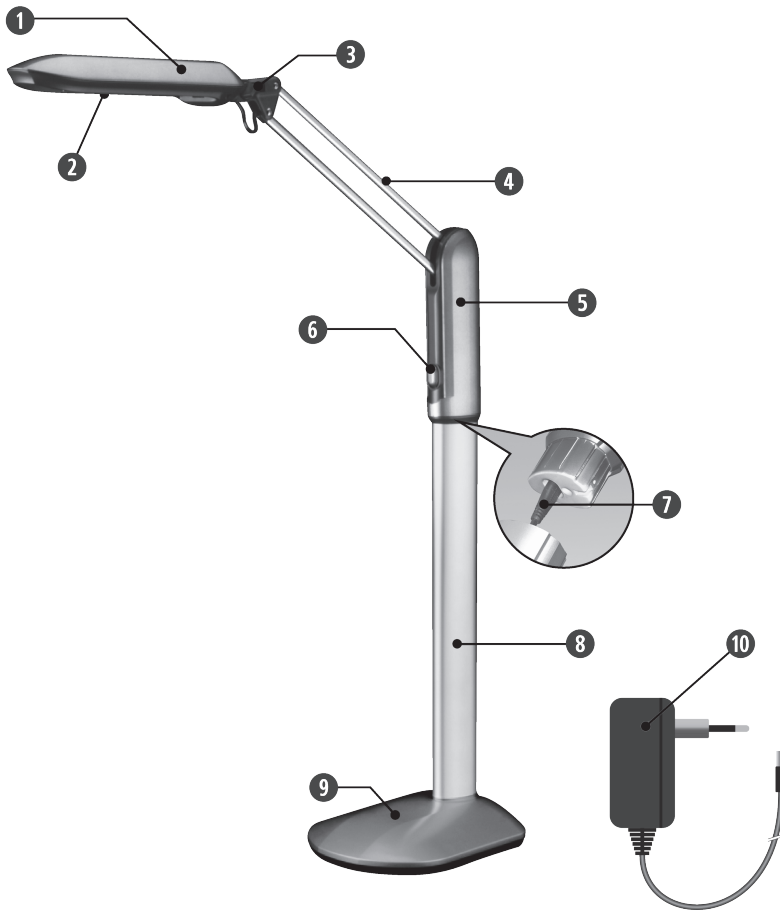Deutsch.....	3
English.....	6
Français	9
Italiano	12
Español.....	15
Nederlands	18
Dansk.....	21
Svenska	24
Norsk.....	27
Suomi.....	30
Polski.....	33
Česky.....	36

Deutsch

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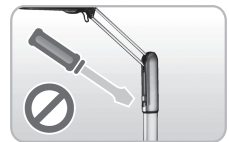
English

Important safety instructions

- Do not use the luminaire in a damp or dusty location.
- Do not use solvents, abrasive cleaners or aerosol cleaners to clean the luminaire. These could cause discolorations.
- If you do not intend to use the luminaire for an extended period, remove the power adapter from the power socket.
- Never open the housing of the luminaire. Never try to modify the luminaire in any way. Otherwise there is a **risk of a fatal electric shock or a risk of fire!** In addition, this will also void the warranty.
- Never place the luminaire in the vicinity of hot objects or in direct sunlight.
- Do not use the luminaire if the power adapter or its cable are damaged. A damaged power adapter cable may only be replaced by the manufacturer or an service centre authorised by it. This appli-
ance may only be operated using the original power adapter.
- Never operate the luminaire in the vicinity of flammable items or materials.


- Do not damage the power adapter cable and do not route the cable in the vicinity of hot objects. Do not kink or twist the power adapter cable. Always pull on the power adapter itself; never pull on the power cable. Do not place any heavy objects on top of the cable. Never connect the power adapter cable to other cables.


Overview of the luminaire components (see fig. on p. 2)

- | | |
|---------------------|----------------------------|
| ① Lampshade | ⑥ Switch |
| ② Light source | ⑦ Plug connection |
| ③ Adjusting screw | ⑧ Aluminium pipe |
| ④ Connecting piece | ⑨ Stand |
| ⑤ Luminaire housing | ⑩ Power adapter with cable |

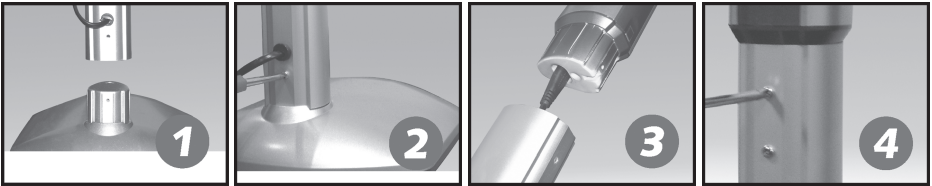Assembling the luminaire

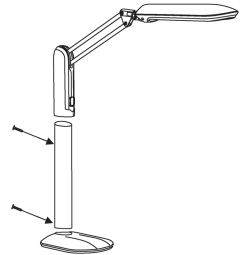
1. Remove all parts of the appliance from the packaging. Place the stand on the floor. Push the aluminium pipe (with the socket downwards and the loose cable end at the top) onto the stump on the stand.
2. Take one of the supplied screws and attach the aluminium pipe to the stand as shown above.
3. Then connect the plug on the cable coming out of the aluminium pipe with the socket on the luminaire housing.
4. Push the base of the luminaire housing into the aluminium pipe and screw the two firmly together.

Operation

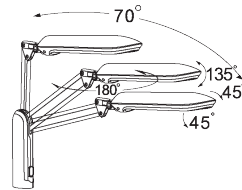- ▶ Set up the luminaire in the desired location taking the safety instructions into account.
- ▶ Connect the cable of the power adapter to the socket on the body of the luminaire. Insert the power adapter into a correctly installed 230 V household power socket.
- ▶ Press the switch to turn the luminaire on/off.
- ▶ Adjust the lampshade to achieve the required lighting effect.
- ▶ Do not use force when adjusting the direction of the lampshade!

Technical data

Item number	16030
Input voltage, power adapter	100 - 240 V ~
Output voltage, power adapter	24 V =
Output	8 W
Luminous flux	550 lm
Illuminance (at a distance of 30 cm)	1900 lx
Colour temperature	4000 K
CRI	80
Energy efficiency class (A - G)	G
LED service life	40,000 hrs.

The light source is permanently installed and cannot be replaced.

Disposal

This product is subject to the European WEEE Directive on waste electrical and electronic equipment in its currently valid version.

Dispose of your product via a certified disposal facility or your municipal disposal facility, not in the household waste. Observe the rules applicable in your country. In case of doubt, contact your disposal facility. Dispose of all packaging materials in an environmentally friendly manner.

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Warranty

Within the framework of the applicable statutory provisions, we provide a warranty for the functioning of the product described in this manual with regard to faults arising in connection with manufacturing errors or material errors. In event of damage due to improper handling, including damage from a fall or impact, no warranty claim exists. Warranty claims are only accepted upon presentation of the sale receipt!

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Eschenbach Optik GmbH